### Deutsch

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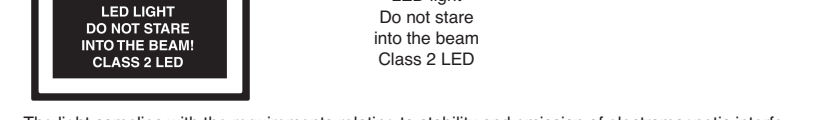
### English

**1. Important information – read prior to start-up**  
You are supplied with a high-quality RIESTER examination light manufactured in compliance with Directive 93/42/EEC for medical products and subject to continuous stringent quality control. Should you have any queries, please contact the Company or your RIESTER Agent who will be pleased to assist you. For additional information please refer to the instructions for use. The address of your authorised RIESTER Agent will be supplied to you on request.  
Please note that correct and safe operation of instruments will only be guaranteed when RIESTER instruments and accessories are used throughout.

**2. Purpose**  
The universally applicable ri-magic® LED light by Riester has been manufactured for lighting and therefore diagnosis and minor surgery in many medical fields, offering a light intensity of approx. 100.000 lux at a distance of 400 mm.

**3. Safety and electromagnetic compatibility**  
Meaning of symbols on power pack and light plinth:

- Attention: Comply with instructions for use!
- Protection class II instrument
- Attention: Do not stare into the beam
- Attention: Used electrical and electronic products are not to be disposed as unsorted municipal waste and are to be collected separately according to national/EU regulations.
- Application part type B
- not for outdoor use
- Alternating current
- Direct current

The light complies with the requirements relating to stability and emission of electromagnetic interference. Please note that interference may not be completely ruled out under the increased influence of detrimental field strengths, i.e. when operating mobile telephones and radiological equipment.

- Note**
- Don't use the examination light for eye-examination
  - There is a risk of gas ignition when the light is used in the presence of inflammable drug mixtures such as air and/or oxygen or laughing gas!
  - The light should only be opened by skilled persons to avoid a risk of lethal electrocution.
  - Please use exclusively the included plug connector  
Manufacturer Glob Tek, Typ GTM 41076  
Possibility of alternative Glob Tek, Typ GTM 21089-1305-T3  
Manufacturer Glob Tek, Typ GTM 41060

### 4. Assembly instructions prior to start-up

**a) ri-magic® LED desk model**  
Attach universal clamp (1) and screw set (2) to a desk or wall rail. Insert universal light (4) from top into the bore of the screw clamp and fix with the knurled bolt (5). Connect cable plug (6) of the power pack (7) to the socket (8) from below until engaging with a click. Connect a transformer (7) to the socket. The light is now ready for operation.

**b) ri-magic® LED wall model**  
Hold wall bracket (1a) to the desired point on the wall and mark 4 holes for dowels. Remove wall bracket and drill the 4 marked holes. Insert dowels into the wall. Apply wall bracket and fix with screws. Insert light (4) into bracket from above and fix by the locking screw (3a) on the side. Connect cable plug (6a) of power pack (7a) to socket (8a) from below until it engages with a click. Connect transformer (7a) to the socket. The light is now ready for operation.

**Caution:**  
The wall must absolutely be solid enough for a good and guaranteed fixation of the diagnostic lamp. Please only use the included screws and anchors.  
Possibility of alternative use of 6 mm joints with lens head screw DIN 95 x 3 x 40 mm.

**c) ri-magic® LED stand model**  
**Montage:**  
1. Please assemble the mobile stand first and thereby take note of the installation instruction attached.  
Connect cable plug (6b) of power pack (7b) to the socket (8b) on the side until an audible click is heard. The light is now ready for operation.

### 5. Start-up and operation

Initially connect the cable plug (6.a,b) to the socket (8.a,b). Connect power pack (7.a,b) to the mains socket. Voltages on the rating plate of the mains adapter must be noted at all times. There are two optional voltages: 230 V and 120 V.

**On-Off**  
Turn the rheostat (9) in clockwise direction. When a click is heard, the light is switched on.  
Turn the rheostat (9) in counter-clockwise direction until a click is heard. The light is now switched off.  
The Diagnostic lamp can also be interrupted from the main power supply by removing the plug connector out of the socket.

**Rheostat**  
The desired light intensity may be set by turning the rheostat (9). Increase light intensity by turning in clockwise direction and decrease by turning in counter-clockwise direction.

**Focusing**  
A focusing ring (10) is provided at the front of the light head for controlling the light cone of the light.

**Note:**

- The flexible arm of the light should not be bent in a too small radius in order to prevent premature material fatigue.
- For normal operation of the light, the bracket (4) and the universal clamp (1) and/or the tripod bracket (17) will heat up around the socket (8.a,b).

### 6. Instruction for care

**General information**  
The goal of cleaning and disinfection of medical products is the protection of patients, users and third persons and conserving the value of the medical products.  
On account of the product design and the used material, no defined limit of maximum processing cycles can be fixed. The lifetime of the medical products depends on their function and on a appropriate treatment of the devices. Before returning faulty products for repair they must have gone through the described processing process.

**Cleaning and disinfection**  
The ri-magic® LED examination light can be cleaned outside with a humid cloth until optical cleanliness is given.  
Wipe disinfection according to the regulations given by the disinfectant manufacturer. Only disinfectants with approved efficiency and in accordance with the national standards can be used.  
Means for cleaning or disinfection may be a soft, possibly lint-free cloth or Q-tips.  
After disinfection the device must be cleaned with a humid cloth to remove all possible deposits of disinfectants.

**Caution!**  
Prior to cleaning or disinfection of the light, disconnect plug from mains socket!  
The items are not permitted for machine cleaning and sterilisation as they will be damaged!

**7. Specification**  
Modell: ri-magic® LED desk/wall/stand light  
Input: 100V- 240V / - 50-60 Hz / 0,5A  
Output: 5V — 1,2A  
LED  
Type of lamp: LED  
Operating Temperature: +10° to +40°C, relative humidity between 30 and 75% (non-condensing)  
Storage: -5° to +50°C, up to 85% relative humidity (non-condensing)  
Weight: Desk and wall model: approx. 2,5 kg  
Stand model: approx. 7,5 kg

**8. Maintenance**  
The instruments and their accessories do not require any specific maintenance.  
Should an instrument have to be examined for any specific reason whatsoever, please return it to the Company or an authorised RIESTER dealer in your area. Addresses to be supplied on request.

**9. Standards**  
Photobiological safety of lamps and lamp systems DIN EN 62471:2008  
IEC 60601-1

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## Warranty

The product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory.

We are therefore pleased to be able to provide a warranty of

### 2 years from the date of purchase

on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.

All defective parts of the product will be replaced or repaired free of charge within the warranty period. This does not apply to wearing parts.

For RT shock-proof, we grant an additional warranty of 5 years for the calibration, which is required by CE-certification.

A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.

Please remember that all warranty claims have to be made during the warranty period. We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.

In case of a warranty claim or repair, please return the RIESTER product with the completed Warranty Card to the following address:

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